

# **ILab** 300 Plus

Clinical Chemistry

Critical Care  
Haemostasis  
Information Systems

*New*  
SOFTWARE

**PRACTICALITY**

**EASE OF USE**

**CONTINUOUS LOADING**

**OPERATOR SAFETY**

**ECONOMY**

**TECHNOLOGY**

**DEDICATED KITS**

**QUALITY**



*A New Plus for Laboratory efficiency*

## A New

### CONTINUOUS LOADING

Cups and different size tubes can be easily accommodated.

64 routine samples can be loaded simultaneously; 14 stat samples, loaded in a dedicated rack, can be run with no interruption of the routine workflow.



### Practicality



### REAGENT LOADING

Up to 35 reagents can be loaded simultaneously; eight sets of racks (35 positions each) can be reprogrammed. Refrigerated and ambient storage ensure optimum on board reagent stability. Reagent tracking using bar-coding checks location and real time inventory.

### Technology

The sampling arm is equipped with capacitive level sensing and accesses sample and reagent positions, the ISE module and the reaction cuvettes.



# Plus for Laboratory efficiency




## Ease of Use

The software managing the system has been designed by users/ for users. A single screen system monitor allows the user to easily keep under control the status of samples, reagents and analysis; a mouse click only is required to start analysis. Let it amaze you with the flexibility allowed by the user configurable system parameters and the empower on-board tools, such as the customized 1 protection system and multilingual interface.

**NEW SOFTWARE**

- Multilingual
- Windows XP
- Reagent bar-code reader
- Anti carryover program

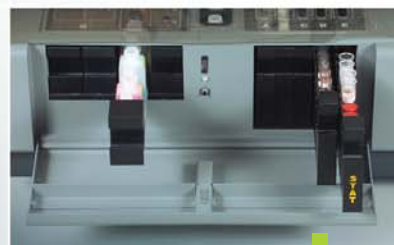

## Continuous Loading

The sample and reagent front loading feature using dedicated racks allows continuous loading without compromising routine workflow, thus maximizing system throughput.



## Operator Safety

The system is designed to allow the user to load and unload samples and reagents in complete safety. Those operations can be performed with no risk to the user of contact with any moving parts.



## Economy

The ILab 300 Plus washes only the cuvettes that have been used with a requirement for only 0,9 L/h of water and detergent. The analyzer does not need to be plumbed into a deionizer system. The semi disposable cuvettes can be individually replaced by the user.

# Plus for Laboratory efficiency



## Dedicated Kits

A wide range of dedicated kits have been developed and validated for use with ILab 300 Plus; this helps to guarantee a high level analytical performance. They have been specifically designed so as to be placed directly on board and to minimize the reagent dead volume. Their size has been defined in order to meet the needs of the routine chemistry laboratory to which ILab 300 Plus system is dedicated.



## Quality



IL applies state of art quality system procedures during design and manufacturing. This assures the compliance of the products with the regulatory requirements applicable to in vitro diagnostics medical devices.

## Methods

The system holds over 150 user definable applications. IL offers a comprehensive menu that includes reagents, calibrators and controls for a wide range of validated applications covering routine chemistry, specific proteins, rheumatology... Available dedicated kits are listed in the tables below.

### ROUTINE CHEMISTRY

Test	Method	Cat. No.	Kit Configuration	Tests/Kit	Calibrators	Formulation
Acid Phosphatase (Total & Non Prostatic)	$\alpha$ - naphthylphosphate	00018482300	5x6 (NP) + 5x6 (T) ml	200	ReferrL G	Liquid
Albumin	Bromocresol Green	00018481800	8x10 mL	250	ReferrL G	Liquid
Alkaline Phosphatase	IFCC	00018480600	16x6.5 mL	400	Factor	Liquid
ALT/GPT	IFCC	00018480700	6x26 + 6x6.5 mL	696	Factor	Dry
ALT/GPT	IFCC	00018257440	8x48 + 4x24 mL	1714	Factor	Liquid
Amylase	CNP-Maltotrioxide	00018481500	12x8 mL	320	Factor	Liquid
AST/GOT	IFCC	00018480800	6x26 + 6x6.5 mL	696	Factor	Dry
AST/GOT	IFCC	00018257540	8x48 + 4x24 mL	1714	Factor	Liquid
Calcium	o-cresolphthalein	00018482800	6x30 + 6x30 mL	600	ReferrL G	Liquid
Cholesterol	Chol. Esterase Chol. Oxidase / Trinder	00018480200	6x35 mL	656	ReferrL G	Liquid
Cholinesterase	BTC	00018482000	6x10 + 4x30 mL	363	Factor	Dry
CK	IFCC	00018482100	8x10 + 4x5 mL	320	Factor	Liquid
CK-MB	Immuno-inhibition	00018482200	6x10 + 6x2.5 mL	250	Factor	Liquid
Creatinine	Jaffè	00018480900	6x16 + 6x16 mL	640	ReferrL G	Liquid
Direct Bilirubin	Modified Jendrassik-Grof	00018481100	4x33 + 8x10 mL	471	ReferrL G	Liquid
$\gamma$ -GT	IFCC	00018480300	4x17 mL	226	Factor	Dry
$\gamma$ -GT	Szasz / IFCC	00018257640	6x40 + 6x10 mL	1000	Factor	Liquid
Glucose	Glucose Oxidase/Trinder	00018480000	6x35 mL	700	ReferrL G	Liquid
HDL Cholesterol	Direct Homogeneous	00018481400	4x30 + 4x10 mL	400	ReferrL HDL	Liquid
LDL Cholesterol	Direct Homogeneous	00018482700	6x9 + 6x3 mL	180	ReferrL LDL	Liquid
Iron	Ferene	00018481700	4x30 + 4x2.5 mL	400	ReferrL G	Liquid
Magnesium	Xlilidyl Blue	00018481600	8x10 mL	228	ReferrL G	Liquid
Phosphorus	Phosphomolybdate Complex	00018481900	6x10 + 6x10 mL	400	ReferrL G	Liquid
Total Bilirubin	Modified Jendrassik-Grof	00018481000	4x33 + 8x10 mL	440	ReferrL G	Liquid
Total Protein	Biuret	00018481300	6x30 mL	600	ReferrL G	Liquid
Triglycerides	GPO/Trinder	00018480500	6x30 mL	600	ReferrL G	Liquid
Urea	Urease/GLDH	00018480400	8x26 + 8x6.5 mL	866	ReferrL G	Liquid
Uric Acid	Uricase/Trinder	00018480100	6x33 mL	660	ReferrL G	Dry
Uric Acid	Uricase/Trinder	00018257740	10x50 mL	1666	ReferrL G	Liquid

### IMMUNOTURBIDIMETRY

Test	Method	Cat. No.	Kit Configuration	Tests/Kit	Calibrators	Formulation
Quantex CRP	Latex-immunoturbidimetry	T3000-2321	4x13 + 4x6.5 mL	162	CRP Standard	Liquid
Quantex ASO	Latex-immunoturbidimetry	T3000-2319	4x13 + 4x2 mL	160	ASO Standard	Liquid
Quantex RF	Latex-immunoturbidimetry	T3000-2320	4x13 + 4x2 mL	160	RF Standard	Liquid
Quantex Ferritin	Latex-immunoturbidimetry	T3000-2328	8x4 + 2x4 mL	129	Ferritin standard	Liquid
Quantex IgA	Immunturbidimetric	T3000-2322	1x30 + 4x6 mL	96	Protein standard	Liquid
Quantex IgG	Immunturbidimetric	T3000-2323	1x30 + 4x6 mL	96	Protein standard	Liquid
Quantex IgM	Immunturbidimetric	T3000-2324	1x30 + 4x5 mL	100	Protein standard	Liquid

### CALIBRATORS

	Description	Cat. No.	Kit Configuration	Parameters	Composition
Routine Chemistry	ReferrL G	00018257000	10x3 mL	Substrates and Electrolytes (no ISE)	Lyoph. bovine based
	ReferrL HDL Cholesterol	00018255800	2x3 mL	HDL Chol	Lyoph. human based
	ReferrL LDL Cholesterol	00018256100	2x1 mL	LDL Chol	Lyoph. human based
	ISE CALIBRATOR A	00018469700	10x100 mL	Na, K, Cl	Acqueous Standard
	ISE CALIBRATOR B	00018469800	4x20 mL	Na, K, Cl	Acqueous Standard
Immunochemistry	Quantex RF Plus standard	T3000-2252	6x1 mL	RF	Liquid human based
	Quantex Ferritin standard multipoint	T3000-2223	5x1 mL	Ferritin	Liquid human based
	Quantex CRP Plus standard	T3000-2093	6x1 mL	CRP	Liquid human based
	Quantex ASO Plus standard	T3000-2312	6x1 mL	ASO	Liquid human based
	Quantex Protein standard multipoint	T3000-2128	5x1 mL	IgA, IgG, IgM, alpha1-AGP, Transferrin	Liquid human based

### CONTROLS

	Description	Cat. No.	Kit Configuration	Parameters	Composition
Routine Chemistry	SeraChem Control Level 1	00018162412	12x5 mL	Routine Chemistry	Lyoph. human based
	SeraChem Control Level 2	00018162512	12x5 mL	Routine Chemistry	Lyoph. human based
Immunochemistry	Quantex ASO-CRP-RF control I	T3000-2069	6x1 mL	ASO, CRP, RF	Liquid human based
	Quantex ASO-CRP-RF control II	T3000-2070	3x1 mL	ASO, CRP, RF	Liquid human based
	Quantex Ferritin/Myoglobin/IgE control I-II	T3000-2222	2x3 mL	Ferritin	Liquid human based
	Quantex Protein control I-II	T3000-2122	3x1 + 3x1 mL	IgA, IgG, IgM, alpha1-AGP, Transferrin	Liquid human based

The ILab Test Menu also includes: Glucose Hexokinase, Lipase, LD-LDH,  $\alpha$ 1 Acid Glycoprotein, Soluble Transferrin, Transferrin, Immunoglobulin E, Myoglobin, Complement 3, Complement 4, Microalbumin, Haptoglobin, HbA1c. For detailed information, please, refer to ILab 300 Plus Application Manual P/N 00018461800.

## Technical Specifications

Tests on board	Up to 35 + 3 ISE
Test Menu	Open System, over 150 Photometric + ISE
Throughput	Up to 200 photometric tests/hour + up to 160 ISE tests/hour
Measuring Methods	End Point, Fixed Time, Kinetic
Carry over management	Yes. Special wash and sampling order <b>New</b>
Sample Loading	Rack : 64 routine (continuous loading), + 14 STAT positions, + 14 Cal&QC positions
Sample containers	Cup, micro cup, Primary tubes (diam. 10 -16, height 50 - 100 mm)
Sample volume	From 3 to 99 $\mu$ L (step 1 $\mu$ L)
Reagent dispensing system	Direct System, 1 to 3 reagents/test
Reagents on Board	35 (+ 1 ISE); 20 refrigerated
Reagent Volume	From 25 to 400 $\mu$ L (step 1 $\mu$ L)
Reaction cuvette	60 semi disposable optical cuvettes, system laundry
Reaction cuvette volume	min. 290 $\mu$ L max. 600 $\mu$ L
Deionized water and diluent usage	About 4 mL/test
Wavelengths	8, range 340 to 620 nm.
Host Interface	Bi-directional RS-232; host query or batch mode
<b>Installation Requirements</b>	
Power	100-230 VAC, 47-63 Hz, 300 W
Temperature / Relative Humidity	18-30 °C , 20-85 % RH non condensing
Dimensions and weight	100 cm width, 65 cm depth, 42 cm height, 65 Kg
Operative System / Printer	Windows XP Professional / Laser printer <b>New</b>
<b>Options</b>	
Sample Bar-code reader, Reagent Bar-code reader, ISE module (Na, K, Cl) <b>New</b>	

## Worldwide Locations

**Instrumentation Laboratory  
Corporate Headquarters**  
Barcelona, Spain  
Tel. +34-93-4010101  
www.ilww.com

**US, Canada, Latin America  
Headquarters**  
Lexington, MA, U.S.A.  
Tel. +1-781-861-0710  
www.ilus.com

**Mexico**  
Col. Granada  
Tel. +52-55-5262-1760  
www.il-mexico.com.mx

**USA**  
Lexington, MA  
Tel. +1-781-861-0710  
www.ilus.com

**Pacific Headquarters**  
Minato-ku, Tokyo, Japan  
Tel. +81-3-3437-6350

**Hong Kong**  
Hong Kong  
Tel. +852-27927773

**Japan**  
Minato-ku, Tokyo  
Tel. +81-3-3437-6350

**Europe, Middle East,  
Africa Headquarters**  
Milano, Italy  
Tel. +39-02-25221

**Austria**  
Wien  
Tel. +43-1-2565800-0

**Belgium**  
Zaventem  
Tel. +32-2-7252052  
www.il-be.com

**Czech**  
Praha  
Tel. +420-2-7816047

**France**  
Paris  
Tel. +33-1-53338600  
www.il-france.fr

**Germany**  
Kirchheim bei München  
Tel. +49-89-909070  
www.ilger.de

**Hungary**  
Budapest  
Tel. +36-1-4392910 or 11

**Italy**  
Milano  
Tel. +39-02-25221  
www.il-italia.it

**Lithuania**  
Kaunas  
Tel. +370-37-313157

**Poland**  
Warszawa  
Tel. +48-22-3361800

**Russia**  
Moscow  
Tel. +7 495 9823723

**The Netherlands**  
Breda  
Tel. +31-76-5480100  
www.il-nl.com

**United Kingdom**  
Warrington, Cheshire  
Tel. +44-1925-81-0141  
www.il-uk.com

This Brochure is used throughout the world.  
Applications / tests listed may not yet be approved by the regulatory authorities in your country.  
Please contact the appropriate office above for specific information regarding availability in your country.  
IL product specifications are subject to modification to assure the highest quality performance.  
Some of the IL sites may still be in the process of completing ISO.  
© Instrumentation Laboratory 2005