MEDI-CAL-U

MULTIPARAMETRIC CLINICAL CHEMISTRY URINE CALIBRATOR

For discrete analyzers

Product code: 1579-0185

Package: 6 x 15 ml

Store at: 2 – 8°C

For *in vitro* use

INTENDED USE

Calibration material for the preparation of calibration curves in human urine using MEDICON reagents on MEDILYZER®, Diatron Pictus series, BECKMAN COULTER AU400/600/600-IVD/640/2700/5400, ADVIA 1200/1800/2400, DIMENSION® R_{xL} /R_{xL} HM /R_{xL} Max/R_{xL} Max HM/ Xpand/ Xpand HM/Xpand Plus/Xpand Plus HM/ EXL/EXL LOCI Module, and every other type of automated biochemical analyzer. For *in vitro* use only.

COMPOSITION

Liquid calibrator based on human urine with additional salivary amylase, hCG of human urine origin, bovine serum albumin and stabilizers.

It contains the following analytes: Amylase, Calcium, Creatinine, hCG, Total Protein, Magnesium, Microalbumin, Urea, Uric Acid, Glucose, Phosphorus, Potassium, Sodium and Chloride.

WARNINGS - PRECAUTIONS

- For *in vitro* use only.
- Avoid contact with eyes, skin and mucous membranes.
- Every donor used for the preparation of this control was tested and found negative for HbsAg, anti-HCV, anti-HIV 1 and 2 by FDA approved methods. However since no test method can offer complete assurance that infectious agents are absent, this product should be handled as any potentially infectious material.

PREPARATION

Invert vials several times to ensure a homogeneous mix. Avoid foaming. Use necessary quantity of the calibrator. Note the date the calibrator is unsealed on the vial. Replace cap immediately after use and store at 2-8°C. Unsuitable storage, handling, or errors during the analytical procedure may give erroneous results.

STORAGE - STABILITY

Sealed, the calibrator is stable at 2-8°C, up to the expiry date stated on the label. After opening, vial contents remain stable up to the expiry date stated on the label, if after each use the vial is tightly capped and stored at 2-8°C. Do not freeze.

It is possible that Creatinine values may drop during the product's life-cycle. In that case, contact MEDICON.

DETERIORATION

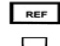
The calibrator should not be used:

- After prolonged exposure to direct sunlight or high temperature.
- After the expiration date.
- When microbial growth is evident.

WASTE

Dispose of all waste material in accordance with local guidelines. Safety data sheet is available for professional use on request.

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